Nutri-Bact

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Cetrimide Agar is a solid selective medium used for the isolation and identification of *Pseudomonas aeruginosa*. Cetrimide inhibits bacterial growth except *P. aeruginosa* and enhances fluorescein and pyocyanin pigment production.

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This medium is for *in vitro* diagnostic use only.

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Store prepared media between 2-8°C, protected from direct light. Store dehydrated media in a dry place, in its tightly-sealed original container between 2-25°C.

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Media should not be used if the expiry date has passed. Prepared media should not be used if there are signs of contamination or deterioration (shrinking, cracking, evaporation or discoloration). Do not use dehydrated media if it is caked.

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Suspend **45,3 g** of dehydrated medium in a final volume of 1000 ml distilled water. Heat gently with frequent agitation to dissolve completely and boil for one minute. Add 10 ml of glycerol. Autoclave at 121°C for 15 minutes. Cool to 45-50° C. Mix gently and dispense into sterile Petri dishes.

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Prior to inoculate, the prepared media should be brought to room temperature.

1. Directly, inoculate plates with specimens and streak for isolation. Tubes used for identification, should be inoculated with a 24 hours growth.
2. Incubate at 35 ± 2°C for 24-48 hours.
3. Incubate for an other 24 hours if the growth is negative.

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Results after 24 hours at 35 ± 2°C aerobically.

Souches / Strains	ATCC	Croissance / Growth	Pigment
<i>Pseudomonas aeruginosa</i>	27853	+	+
<i>Pseudomonas putida</i>	12633	+	-
<i>Stenotrophomonas maltophilia</i>	13637	-	-
<i>Escherichia coli</i>	25922	-	-

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This medium allows partial identification. Additional biochemical tests, as well as morphological characteristics and serological typing, may be required.

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1. Brown, V.I., Lowbury, E.J.L. 1965 Use of an improved cetrimide agar medium and of culture methods for *Pseudomonas aeruginosa*. J. Clin. Pathol. 18: 752.
2. King, E.O., M.K. Ward, D.E. Raney. 1954 Two simple media for the demonstration of pyocyanin and fluorescein. J. Lab. Clin. Med. 44:301-307.
3. Lennette, E.H., Ballows, A., Hausler, W.J.Jr., and Shadomy, H.J. Manual of Clinical Microbiology. 4th ed. 1985 Washington D.C.: American society for Microbiology.
4. Mac Faddin, Jean F., Medi for Isolation-Cultivation-Identification-Maintenance of Medical Bacteria Vol.1 1985. Baltimore, MD. Williams & Wilkins.
5. Washington, J.A. Laboratory Procedures in Clinical Microbiology. 1981. Springer-Verlag, New York.

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