Nutri-Bact

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Kligler Agar is a differential medium by ability to determine carbohydrate (dextrose and lactose) fermentations and hydrogen sulfide production especially of enteric pathogens. Phenol Red is the pH indicator and changes from red to yellow with a positive result. The butt medium blackening indicates presence of H₂S.

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This medium is for *in vitro* diagnostic use only.

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Store prepared media between 2-8°C, protected from direct light. Store dehydrated media in a dry place, in its tightly-sealed original container between 2-25°C.

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Media should not be used if the expiry date has passed. Prepared media should not be used if there are signs of contamination or deterioration (shrinking, cracking, evaporation or discoloration). Do not use dehydrated media if it is caked.

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Suspend **55.0 g** of **Kligler Agar** powder into a final volume of 1000 ml of filtered deionized water. Heat with frequent agitation in order to dissolve completely, then let boil for one minute. Dispense 10 ml into culture tubes and sterilize for 15 minutes at 121°C. Allow the tubes medium to set in sloped form with a butt about 5 cm high.

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Prior to inoculate, the prepared media should be brought to room temperature.

1. Inoculate with 2-3 similar colonies and stab the butt to about 0,5 cm from the bottom of the tube.

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2. Incubate at 35°C for 24 heures with loosened caps.
3. Examine for reactions.

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Results after 24 hours at 35°C aerobically.

Souches Strains	ATCC	Croissance & Phénotype / Growth & Phenotype			
		Pente / Slant	Culot / Butt	Gaz / Gas	H ₂ S
<i>Escherichia coli</i>	25922	A	A	+	-
<i>Proteus mirabilis</i>	43071	A	A	-	+
<i>Pseudomonas aeruginosa</i>	27853	K	K	-	-
<i>Salmonella typhimurium</i>	14028	K	A	+	+

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This medium is only a part of the identification. Other tests may be required.

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1. Ewing, W.H. 1985 Edwards and Ewing's identification of Enterobacteriaceae. 4th ed Elsevier Science Publishing Co. Inc. New York.
2. Lennette, E.H., Ballows, A., Hausler, W.J.Jr., and Shadomy, H.J. Manual of Clinical Microbiology. 4th Ed. 1985 Washington D.C.: American society for Microbiology.
3. Mac Faddin, Jean F. 1985 Media for Isolation-Cultivation Identification-Maintenance of Medical Bacteria. Vol. I, Baltimore, MD.: Williams & Wilkins.
4. Washington, J.A., 1981 Laboratory Procedures in Clinical Microbiology. Springer-Verlag, New York.

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