Nutri-Bact

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MacConkey Agar is a selective medium for isolation and differentiation of lactose fermenting coliforms from lactose non-fermenting enteric pathogens. Crystal violet and bile salts, inhibit the growth of Gram-positive organisms. Lactose fermentation by coliform acidifies medium, precipitate bile salts and produce deep red colonies. Colourless colonies are lactose non-fermenting organisms.

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This medium is for *in vitro* diagnostic use only.

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Store prepared media between 2-8°C, protected from direct light. Store dehydrated media in a dry place, in its tightly-sealed original container between 2-25°C.

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Media should not be used if the expiry date has passed. Prepared media should not be used if there are signs of contamination or deterioration (shrinking, cracking, evaporation or discoloration). Do not use dehydrated media if it is caked.

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Suspend **50,0 g** of dehydrated medium in a final volume of 1000 ml distilled water. Heat gently with frequent agitation to dissolve completely and boil for one minute. Autoclave at 121°C for 15 minutes. Cool to 45-50° C. Mix gently and dispense into sterile Petri dishes.

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Prior to inoculate, the prepared media should be brought to room temperature.

1. Inoculate the plates with specimen and streak for isolation.
2. Incubate for 24-48 hours at 35 ± 2°C.

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Results after 24 hours at 35 ± 2°C aerobically.

Souches / Strains	ATCC	Croissance / Growth	Couleur / Colour
<i>Escherichia coli</i>	25922	+	Rouge / Red
<i>Escherichia coli</i>	35150	+	Rouge / Red
<i>Proteus mirabilis</i>	12453	+	Incolore / Colourless
<i>Salmonella enterica</i>	14028	+	Incolore / Colourless
<i>Enterococcus faecalis</i>	29212	-	
<i>Staphylococcus aureus</i>	25923	-	

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This medium allows partial identification. Additional biochemical tests, as well as morphological characteristics and serological typing, may be required.

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1. Lennette, E.H., Ballows, A., Hausler, W.J.Jr. and Shadomy, H.J. 1985. Manual of Clinical Microbiology. 4<sup>th</sup> ed. Washington D.C.: American society for Microbiology.
2. N.C.C.L.S. 1990. Quality Assurance for Commercially Prepared Microbiological Culture Media. Approved Standard. Vol.10, No.14. NCCLS Document M22-A.
3. Mac Faddin, J.F. 1985. Media for Isolation-Cultivation-Identification-Maintenance of Medical Bacteria. Vol.1. Baltimore, MD.: Williams & Wilkins.

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