Nutri-Bact

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Mueller Hinton agar is a solid medium used for susceptibility testing of pathogens. Bauer *et al.* have recommended this medium as routine methods for disk susceptibility testing. Disks impregnated with known amount of antibiotics are placed on the agar surface. Trimethoprim/sulfomethoxazole disk gives a satisfactory clear zone with *Enterococcus faecalis* ATCC 29212 if thymidine level is as low as possible.

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This medium is for *in vitro* diagnostic use only.

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Store prepared media between 2-8°C, protected from direct light. Store dehydrated media in a dry place, in its tightly-sealed original container between 2-25°C.

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Media should not be used if the expiry date has passed. Prepared media should not be used if there are signs of contamination or deterioration (shrinking, cracking, evaporation or discoloration). Do not use dehydrated media if it is caked.

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Suspend **38.0 g** of **Mueller-Hinton Agar** powder into a final volume of 1000 ml of filtered deionized water. Heat with frequent agitation in order to dissolve completely, then let boil for one minute. Autoclave at 121°C, 15 p.s.i. during 15 minutes. Once sterilized, cool down to 45-50°C. Mix gently and dispense aseptically.

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Prior to inoculate, the prepared media should be brought to room temperature.

1. Into a Mueller-Hinton broth with 4-5 similar colonies from an overnight growth, prepare a suspension and adjust to 0.5 McFarland Standard.
2. Following established procedures, inoculate plates by swabbing the inoculum on the surface three times.
3. Place appropriate susceptibility disk on the surface and press gently.
4. Incubate at 35°C for 24 hours.

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Results after 24 hours at 25°C when aerobically grown.

Souches / Strains	ATCC	Clinical Laboratory Standards Institute
<i>Enterobacter faecalis</i>	29212	OK
<i>Escherichia coli</i>	25922	OK
<i>Pseudomonas aeruginosa</i>	27853	OK
<i>Staphylococcus aureus</i>	25923	OK

Rencontre les standards de la Clinical Laboratory Standards Institute. / Meets the Clinical Laboratory Standards Institute standards.

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Disk diffusion testing with Mueller-Hinton agar has been standardized with rapid growing microorganisms. This medium is only a part of identification. Other tests may be required.

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1. Bauer, A.W., W.M.M. Kirby, J.C. Sherris, M. Turck. 1966 Antibiotic susceptibility testing by a standardized single disk method. Am. J. Clin. Pathol. 45:493-496.
2. D'Amato, R.F., L. Hochstein 1982 Evaluation of a rapid inoculum preparation method for agar disk diffusion susceptibility tests J. Clin. Microbiol. 15:282-285.
3. Lennette, E.H., Ballows, A., Hausler, W.J.Jr., and Shadomy, H.J. Manual of Clinical Microbiology. 4th Ed. 1985 Washington D.C.: American society for Microbiology..
4. Munro. S., 1992 Disk diffusion susceptibility testing In; Clinical Microbiological Procedures Handbook. Hindler J., section ed; Isenberg HD ed. Washington D.C. American Society for Microbiology 5.1.1-5.1.30.
5. NCCLS 1990 Performance Standards for Antimicrobial Disk Susceptibility Tests. Approved Standard 4th Ed. Document M2-A4 Vol.10, No.7.
6. NCCLS 1994 Performance Standards for Antimicrobial Susceptibility Testing 5th Informational Supplement Document M100-S5 Vol.14, No.16.
7. Pollock, H.M., A.L. Barry, T.L. Gavan, P.C. Fuchs, S. Hansen, C.L. Thornsberry, H.Frankel, S.B. Forsythe 1986 Selection of a Reference Lot of Mueller Hinton Agar J. Clin. Microbiol. 24:1-6.

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